CARTE DE GARANTIE / WARRANTY CARD / GARANTIEKARTE / GARANTIE
12 MOIS / 12 MONTHS / 12 MONATE / 12 MAANDEN

TENSIOMETRE

Ref. WE100

NOM/NAME/NAHME/NAAM _____

ADRESSE/ADRESS/ANSCHRIFT/ADRES _____

CODE POSTAL/POSTCODE/POSTLEITZAHL/POSTCODE _____

VILLE / PLACE / STADT / _____

CACHET DU REVENDEUR
STAMP OF RETAILER
STEMPEL VOM HÄNDLER
STEMPEL VAN DE VERKOPER

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Wrist Blood Pressure Monitor

1.1.Features:

- One touch. Accurate result in 20 seconds.
- Comfortable automatic inflation and deflation.
- Memories for reading up to 99 sets, with date/ time
- Automatic turn-off after idle 1 minute.

1.2. Contraindications:

- Do not use the device on infants or persons who cannot communicate.
- The device may show some reading error in the presence of common arrhythmia, such as arterial or ventricular premature beats or arterial fibrillation.
- Do not use the device for purposes other than measuring blood pressure.

1.3. Precautions:

- This unit does NOT serve as a cure of any cardiovascular symptoms or disease. The data measured is only for reference. Always consult your physician to have the results interpreted. Do not rely on self-diagnosis.
- Always follow the operation procedures described in the manual in order to obtain correct results.
- Do not press the power button when the wrist cuff is not wrapped around the wrist.
- Using the device near a mobile phone or microwave oven may cause a reading error.
- Disassembly or modification the unit or wrist cuff may lead to permanent reading error.

Suggestions before measuring blood pressure:

- Avoid eating, smoking, exercising, bathing, and taking alcohol, caffeine, or tea for at least 30 minutes before measurement.
- Avoid urination or bowel movement before measuring.
- Avoid measuring when feeling anxiety or tension.
- Please sit or lie down for at least 5~10 minutes before measuring.

2. DESCRIPTION FO UNIT (Schema p2)

1. The date/time
2. Systolic Pressure
3. SET button to set the date/time and the memory
4. Diastolic Pressure
5. Pulse symbol
6. Battery symbol
7. Power button: Press to start inflating the cuff and measuring the blood pressure & pulse rate. Press again to turn off the device.
8. Memory button: To set the Date/ Time and to the stored measurements.
9. Memory symbol

3. PREPARATIONS BEFORE USE

3.1. Battery installation / replacement

1. Remove the cover in the direction indicated by arrow (fig.2 p.2)
2. Install or replace 2 "AAA" size alkaline batteries. Make sure the + (positive) and - (negative) marks match those indicated in the battery compartment.
3. Replace the battery cover.

Note: For best performance and longest life, we recommend using alkaline batteries. Do not mix difference type, date or brand-name batteries.

This device features 2 setup battery warning:

- a. Battery low warning: battery indicator  appears on LCD display. It means the battery is running low. You can still make about accurate measurements.
- b. Battery dead warning: when the battery indicator  flashes, the battery is dead. Please change immediately.

3.2. Date, Time, and Unit Setting

The SET button is on the top right of the LCD screen display (n°3 - Fig.1 - p.2).

Press the SET button with the tip of a pen to access the Date/Time setting menu.

Press the SET button for 4 secondes to set minutes, short press a second time to set the month, short press a third time to set the year, and short press a fourth time to set hours. For each information, make the values scroll pressing the arrow key (n°8 - Fig.1 - p.2) and press the SET button to validate.

3.3. Clear the memory

Press the arrow key (touche n°8 - Fig.1 - p.2), then press the SET button for 3 secondes to make the symbol «CLR» appear on the LCD screen display.

3.4. Pulse function

There are two warning signs to display with the Pulse rate if the user's pulse rate is high or low.

If it shows «PH», it means «Pulse High»

If it shows «PL» it means «Pulse Low»

In both case, please consult your doctor.

4. How to measure blood pressure

Hold your left arm in front of you with palm facing up.

Apply the cuff so that the monitor is on the inside of your wrist with the display in easy view (fig3- p.2). Place your elbow on a table, palm facing up, Reset your wrist on the case to make sure the pressure cuff is at the same level as the heart. (fig4 p2)

2. Press "power" button and relax (n°7 fig1- p.2).

Note: Please take a 5-10 minute break before taking another measurement.

5. HOW TO USE THE MEMORY FEATURES

5.1 Memory mode

The blood pressure and pulse rate readings measured are automatically stored in the memory. The unit stores the most recent 99 sets of readings. When more than 99 measurements were taken, the earliest set will be replaced by the most recent. Use the memory function to recall previous readings.

Press the ARROW button (n°8 - Fig.1 - p.2), the index "1" will be displayed, followed by the last reading taken. (figure 10).

To recall the readings measured previously, press "memory" button again. The values will be displayed in the order of time and date taken.

5.2 How to erase all the readings in memory

Press the ARROW key (n°8 - Fig.1 - p.2) then the SET button (touche n°3 - Fig.1 - p.2) during 4 secondes, until «CLR» appears on the top left of the display.

Then choose to clear one memory or all the memory pressing a second time on the SET button.

Validate your choice pressin the ARROW key.

6. CARING FOR YOUR METER

6.1 Maintenance

Storage

- Please remove the batteries if the monitor won't be used for a long period of time.
- Always use the storage box provided with the device to transport or store the unit.
- Store the unit in a dry and cool place within the storage condition range. The device shelf life is 3 years in normal condition except for battery. Do not operate or store the device in direct sunlight, high temperature, humid, dusty, or corrosive environment.
- Please refer to "specification" for the proper storage and operation environment.
- Do not drop this unit. Avoid strong impact.

Cleaning

- Do not use benzene, gasoline, or any alcoholic-base or solvent agent to clean the device.
- To clean the unit, wipe with a cloth moistened with tap water or a mild cleaning agent, then dry the device with a soft and dry cloth. Do not flush with or immerse in water.
- The cuff can be cleaned by wiping with moistened cloth and soap. Do not immerse in water.
- Do not twist any tubing. Refrain from stretching, bending or tugging on the wrist cuff.
- Do not flip the cuff inside out.
- Do not separate the cuff from the unit, disassemble the device, or try to repair the unit by yourself. If any problem occurs, refer to the distributor.

6.2. Error message & troubleshooting

Case n°1: No indicator/symbol on the LCD;

Batteries ran out or were not installed properly : Check and correct the battery polarity. Remove the batteries and wait 10 seconds before replacing the batteries.

Case n°2: «Err P» shown on the display: Fail to inflate.

Please check the connection on main unit and cuff. If loosened, then connect them tightly. Please check if air tube is broken, then change it.

Case n°3: «Err 1» shown on LCD display: the cuff is loosened. Deflate too fast.

Fasten the cuff, start it again. Please send it to customer service.

Case n° 4: «Err 2» shown on the LCD display, movement occur on the wrist. Interference signal around the BPM.

Keep still after you press the power button. Please remove interfering sources from magnetic material or cell phone.

Case n°5: «Err 3» shown on the display. Incorrect measured result.

Take off your watch or cuff might be placed incorrectly. Please re-wrap cuff in correct position. You might use it incorrectly, such as measure blood pressure on child.

If correct measurement is impossible even after checking the above mentioned points, contact the customer service.

6.3 Specifications

•Display: LCD digital display

•Measurement Method: Oscillometric method

•Measurement Range

Pressure : 0~280mmHg

Pulse rate : 40~195 / minute

•Accuracy / Calibration

Pressure : 3mmHg of reading

Pulse rate : 5% of reading

•Memory Capacity: 99 sets of reading with date and time

•Inflation: Automatic inflation

•Pressure Release: Automatic exhaust valve

•Pressure Detection: Piezo-Resistive silicon pressure transducer

•Power supply: 2x AAA 1.5V alkaline batteries (not included)

•Operation Environment: +10°C ~ +40°C / 30% ~ 85% R.H. maximum

•Storage Environment: -20°C ~ +60°C / 10% ~ 95% R.H. maximum

•Battery Life: Approx. 200 uses (alkaline batteries)

•Cuff Size: Wrist Circumference approx. 135~195mm (5-1/4 ~ 7-3/4 inch)

•Dimensions: 63,5mm x 69mm x 37,5mm

•Weight: 110g (without batteries)

•Accessories: Storage Case, Instruction Manual

*Please understand that specification may be changed without prior notice.

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