# **SPORT-ELEC®**

## **I N S T I T U T**

### **CHAISE DE MASSAGE**

Massage chair

FR

GB

Réf. : MCA-001/B

**SPORT-ELEC® Institut** 31 Rue du Val Breton - BP 35 - 27520 Bourgtheroulde - Infreville - FRANCE  
TEL : + 33 (0)2 32 96 50 50 - FAX : + 33 (0)2 32 96 50 59  
[info@sport-elec.com](mailto:info@sport-elec.com)

## DESCRIPTION

### Elements

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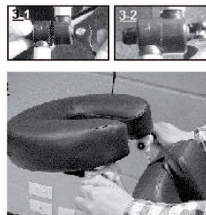
The massage chair provides optimum access to the back and arm. Adjustable adjustable in seconds, it makes work easier and improves comfort.


Aluminum frame, tack on expanded polyester foam, headrest ergonomic shape with an open face, chest plate removable armrests and sloping recumbent, sitting height adjustable hydraulic cylinder, leg-rest, anti-slip feet, 6 adjustment positions

## I - Setting up your table

1. Unfold the chair by pulling the pin height adjustment of the seat and press fermemant on the front foot of the chair

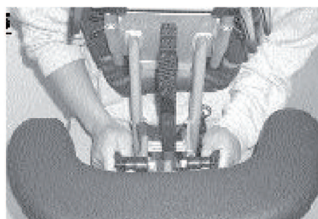2. Adjust the headrest: Open the handle at the base of the headrest and adjust to desired position. Collapse the handle

3. Set the angel of the armrests and pull the safety belt and lock the setting when you convient. Assurez qe the safety belt is properly tensioned to provide protection additional pressure with the arms

4. Adjust the seat height by pushing the button to clear the position on one of 6 levels.

## II - Storage

1. First Fold the headrest and arm rests

2. Ensure sure the button shows the seat adjustment is positioned in the 4th or middle hole (This will allow you to carry and fold more easily.)

3. Turn the chair upright and press the back foot of the massage chair while holding the headrest ( careful not to pinch your hands at this handling)

## **WARNING:**

DO NOT SUPPORT OR ABOVE ADD ADDITIONAL PRESSURE.  
YOU MUST TENDER THE TAPE TO AVOID THE CHAIR COLLAPSE CAUSING SERIOUS ACCIDENTS!

### **Precautionary instructions**

1. Maintain the chair periodically. Make sure all screws are secured and the cable has not deteriorated or frayed. Make sure there are no cracks in the wood structure.
2. Do not overload the chair beyond the recommended weight limit. This could result in serious injury to the user.
3. Do not use alcohol based cleaners when cleaning the vinyl. This could cause damage to the vinyl material. Use warm soapy water and wipe dry.

The Chair can support the weight of a person of 113 kg maximum horizontal position only.

This guarantee is exercised only from the owner of the table and is not transferable.

WHEN YOU SEPARATE YOUR PRODUCT END OF LIFE, BE SURE TO OBSERVE NATIONAL REGULATIONS. THANK YOU AND TO FILE THE CHAIR TO A PLACE PROVIDED FOR THAT PURPOSE TO ENSURE ITS SAFE DISPOSAL OF WAY IN RESPECT OF THE ENVIRONMENT.

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MDE 909

**WARRANTY**

This appliance is guaranteed for 12 months from the date of purchase. During the period of the guarantee, the distributor will take responsibility for defects occur, the appliance will be repaired or exchanged as the distributor deems necessary. Damage arising from improper use or normal wear and defects, which do not materially affect the performance of the appliance, is not covered under this guarantee. The guarantee is not valid if unauthorized repairs have been attempted or if replacement parts used are not approved by the supplier. The guarantee is valid only if it is stamped and signed by the retailer at the time of the purchase. For all claims made during the period of the guarantee, send the appliance and the warranty card to your retailer. All other claims are excluded from the guarantee unless specifically prohibited by law.

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**SPORT-ELEC® Institut SAS-31 Rue du Val Breton - BP 35 - 27520 Bourgtheroulde Infreville-FRANCE**