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## GB UTILITY OF PRODUCT

- Stability of the ankle during activity - Sprain or tendinitis of the ankle
- Protections postoperative - Rehabilitation of ankle ligament.

### MATERIAL

- Neoprene group: 3 mm (neoprene 80 %, nylon jersey 20 %).
- Woven group: Lycra woven textile (55 % polyamide, 45 % elastan).

### IMPORTANT ADVISES AND WARNINGS

This product is not burn protected. It will lose its characteristics by high temperature. Therefore do not use above 248°F. If you want to make changes for your comfort, please ask to your doctor or where you purchased the product. This product can not be used out of its own utility and cannot be combined with other products, only use with its original utilities. Usage of neoprene products is not allowed for more than 3-4 hours. In any case, do not use more than 8 hours (unless otherwise specified by your doctor). Please only use with doctor advice.

### WASHING INSTRUCTIONS

- Please close the velcro bands if there are and take out the locable support (like joint parts...).
- Do not put in washing-machine. Only handwash it at 104°F.
- Do not use bleach.
- No dry-cleaning.
- Do not use dryer.
- No ironing.
- After the wash please be sure that it is totally dry before the use.

### UTILITY OF THE PRODUCT

- Please be sure the size of the product is exactly your size.
- Be sure that bending part of product is exactly placed on the elbow articulation. Close the velcro bands.

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