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Shiatsu Massage Device for the back, buttocks and thighs.

- Relieves muscle tension and relaxes your body
- Massage device with massage heads moving up and down to stimulate the whole back, the upper and the lower parts of the back + vibrations to relax buttocks and thighs
- Imitates the hands of a professional masseur
- Remote control for a comfortable use
- Heat function during for the back massage, from 113 to 122F

The pack contains: (photo I page 1)

1 massager for the back and buttocks (A)

1 Remote control (B)

1 AC adapter (C)

When using this unit for the first time, read carefully the instructions and the following guide-lines in order to become familiar with the controller.

Before the first use: (Photo II page 1)

- 1- use the provided allen wrench to remove the screw by turning counter-clockwise as shown. Dispose of the screw properly
- 2- Place the permanent black plug into the hole from which the screw was removed

How to install the device

1. Plug the AC adapter to a power socket; connect the 12V cable to the controller.
2. Fasten the Shiatsu to your chair or seat for an easy use
3. Sit down comfortably and take the remote

Functions

This device performs kneading massage of the back, buttocks and thighs to relieve the muscle tension.

Unique up-down device.

Multi massage heads to massage in big area.

Combination of kneading for the back and vibrations for buttocks and thighs.

6 kneading heads roll up and down along the spine and knead your entire back.

Three massage variations for the back:

full back, lower back or upper back.

Three intensity levels for vibrations in the seat:

slow, medium, strong.

Suitable at the office, home, or in car (when the engine does not run)

Safety Precautions

This unit is designed to be used by healthy adults. However, the following precautions should be taken when using the unit

Do not use the device in high temperature or humid environment

Avoid the breakage of the power cord

Do not put the cable or plug on any heated surface

Do not dismantle the device yourself. In case of any problem, apply to the Customer Service of Sport-Elec Institut or your retailer.

Do not use the device under a pillow, a blanket, in case of any fire accident.

Do not hit the unit with a sharp article.

Do not use the device out of doors.

Keep this unit out of reach of children.

Do not connect the unit with any other power supply different from that one mentioned at the AC adapter.

Use this appliance only with supplied adapter.

Before undertaking any exercise program, including this unit, consult your family physician to ensure that you are capable of conducting a regular routine without putting undue stress on your body.

If you have never exercised before, if you are pregnant, recently had surgery or suffer from any significant illness, including epilepsy, hemophilia, diabetes, cancer, hypertension, heart disorders, neuromuscular disorders and more, you should not conduct any exercise routine, including this unit, unless under the direction of your physician.

Do not use the device while drowsy or sleeping.

Do not use this device with any other electric equipment at the same time (e.g., another electrical muscle stimulation device). Do not use this unit and any other machine at the same time.

The device is still under tension when the adapter is plugged into a power socket. Please, switch to "OFF" and unplug the device to stop use it.

This unit is neither a medical nor therapeutic device.

Warning

This product is designed to be used only while seated upright. Never use this product in bed

Never use this unit without clothes on

Never cover yourself, or the product, with a blanket or with any other material while using this product

Do not use this product for more than 15 minutes at a time

This product should never be used by individuals suffering from:

- sensory deficiencies in the lower back

- any physical ailment that would limit the user's capacity to operate control.

Never leave this product in use, or plugged in while unattended

Any violation of the above safeguards may cause the unit to overheat, resulting in serious damage or fire

Start the device : (Picture III p 1)

- 1 Start the device by pressing on the button "Power".

Choose your program of massage: -

- 2 "Demo": this program gives you an outline of all the possibilities of the device.

- 3 "Full ": to mass the whole back, from bottom to the top and from top to bottom, from renal area to shoulders.

- 4 « Upper » : to mass the top of the back and shoulders, from bottom to the top and from top to bottom.

- 5 " Lower ": to mass the bottom of the back and the renal area, from bottom to the top and from top to bottom.

- 6 «Heat»: Back massager heads heaters up to 122F. Press the button again to stop the heat

- 7 :You can also choose the intensity of the seat vibrations for buttocks and thighs with 3 possibilities: "L":

Low; "M": Medium; and "H", Strong.

Every program lasts 15 minutes after which the device switches off automatically.

At the end of your massage session, put the remote control in the case situated on the right of the shiatsu seat, then unplug the AC adapter from the power socket and unplug the 12V cable from the controller.

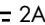
Finally store your Massage Device in the box and keep it in a dry place.

Specification:

Product name : Heating Shiatsu Back Massager

Ref.: MC-18AH

Adaptor: AC100V - 240V/50Hz-60Hz / 30W


DC 12V  2A-

Dimensions : 65 x 39 x 37 cm

Net Weight : 4,5 Kg

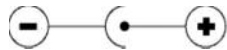RoHS

WARRANTY

This appliance is guaranteed for 24 months from the date of purchase. During the period of the guarantee, the distributor will take responsibility for defects occur, the appliance will be repaired or exchanged as the distributor deems necessary. Damage arising from improper use or normal wear and defects, which do not materially affect the performance of the appliance, is not covered under this guarantee. The guarantee is not valid if unauthorized repairs have been attempted or if replacement parts used are not approved by the supplier. The guarantee is valid only if it is stamped and signed by the retailer at the time of the purchase. For all claims made during the period of the guarantee, send the appliance and the warranty card to your retailer. All other claims are excluded from the guarantee unless specifically prohibited by law.

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